

NOW, IT'S EASIER TO TREAT A BROAD SPECTRUM OF BPH PATIENTS WITH THE EXPANDED UROLIFT® PORTFOLIO¹

Treat the median lobe with confidence. Can be used to treat lateral lobes, including large prostates.*

1 Tissue Control Wings

- Wings hold tissue during manipulation and provide enhanced control
- Shaped to minimize view obstruction

2 Needle Location Marker



- Laser-etched markings can improve targeting accuracy for predictable implant placement
- Needle penetrates tissue inline with the markings

3 Atraumatic Design

- Rounded edges
- Flexible stainless steel material

More tissue control can
reduce procedural manipulations¹

Introducing the
UroLift® ATC™ System

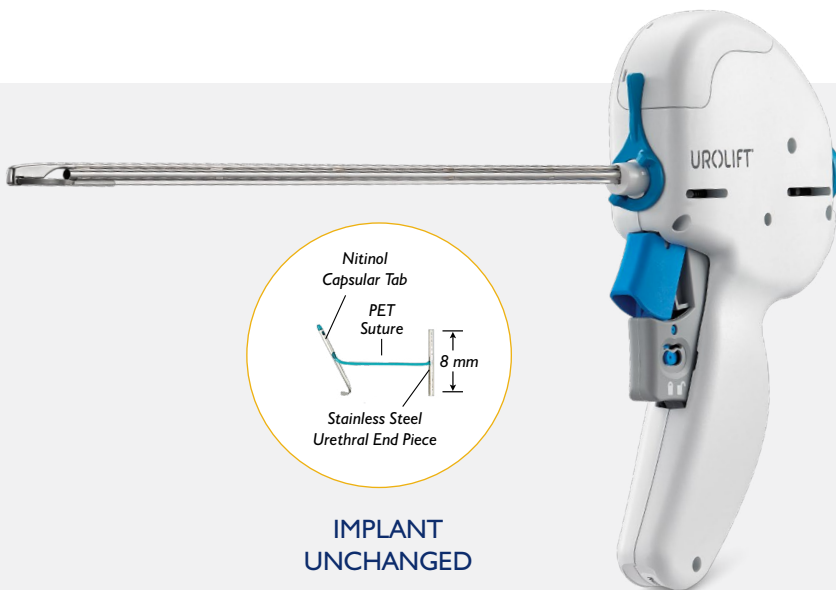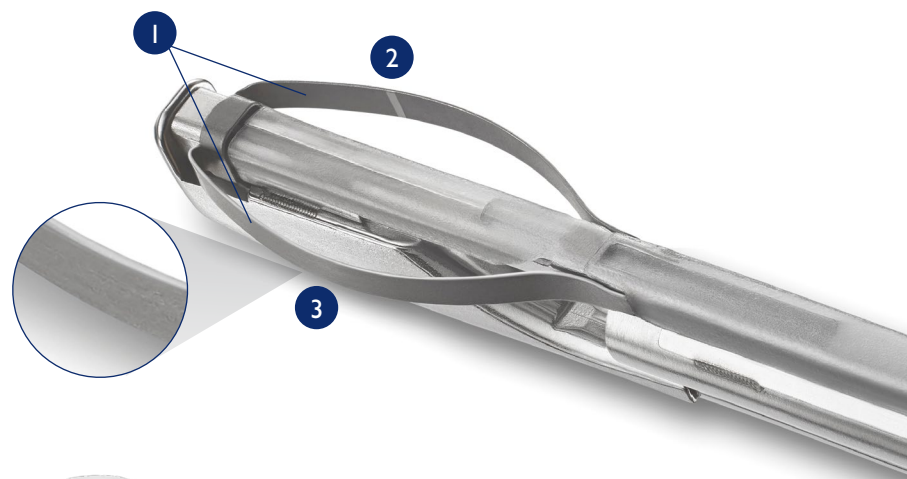

Deliver the same,
proven implant
through an enhanced
delivery device

The UroLift System achieves consistent results in patients with an obstructive median lobe.**

-55%, 13.5pt IPSS↓


at 12 mo. in MedLift Study²

1. Data on file at Neotract; 2. Ruktalis et. al, Prostate Cancer and Prostatic Diseases 2018 MedLift Study


*The UroLift Advanced Tissue Control (ATC) system is indicated to treat patients with prostate sizes up to 100cc.

**Demonstrated with UL400 device in MedLift Study.


TREATING AN OBSTRUCTIVE MEDIAN LOBE WITH THE UROLIFT[®] ATC[™] SYSTEM

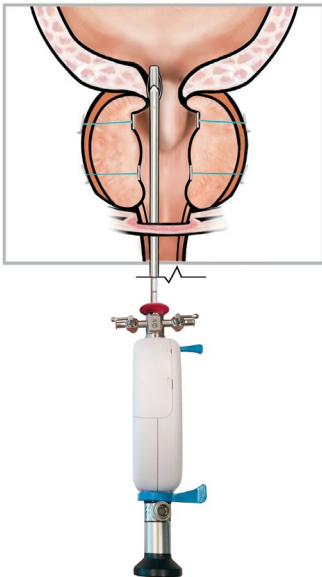


STEP 1:
After deploying lateral lobe implants, roll the median lobe from the bladder into the prostatic urethra and slowly begin compression

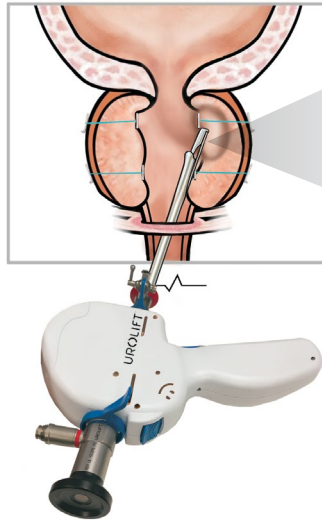


STEP 2:
Compress the median lobe postero-laterally by elevating hands

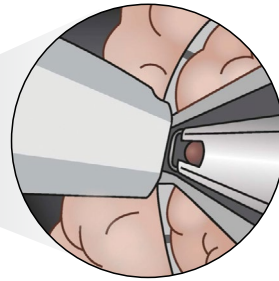


Cystoscopic View
Wings hold tissue, providing enhanced control during the procedure

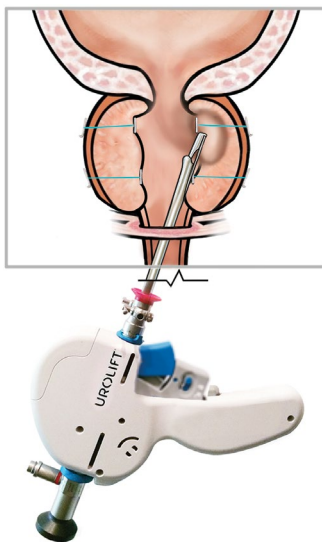


STEP 3:
Slowly lower hands, while maintaining compression

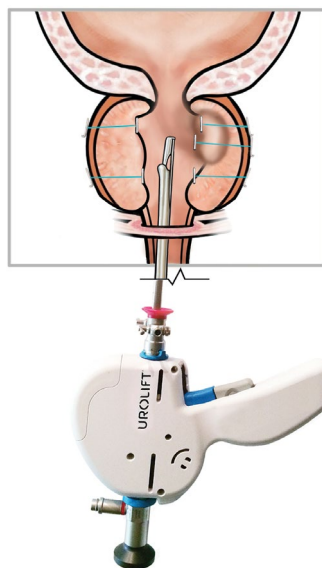

STEP 4:
Deploy implant through the median lobe and treated lateral lobe

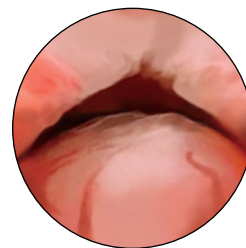

Pre-procedure

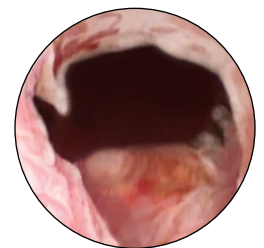

Post-procedure

Results may vary

Please work with your local sales representative for training support.

Refer to the Urolift ATC System Instructions for Use for more information.

Teleflex, the Teleflex logo, NeoTract, the NeoTract logo, Urolift, and Urolift ATC are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries.

© 2020 NeoTract, Inc. All rights reserved. MAC01419-01 Rev A